

○ P6 SCALING TIPS

PLACE YOUR SCALER TIP OVER THE RELEVANT IMAGE TO CHECK YOUR TIP MEASUREMENT

UNIVERSAL TIPS - Designed for supragingival calculus removal.

Each tip is designed for different cleaning needs, from coarse to fine depuration, and for tackling both large deposits and interdental areas.



Up to 1 mm:
Optimum
efficiency



1-2 mm:
Time to
re-order



Over 2 mm:
Time to
dispose


Universal (Coarse)

U6S01



Universal (Fine)

U6G3

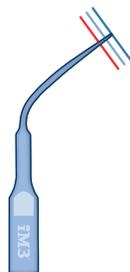

- 🔗 Reorder new tips at www.im3vet.eu
- ⚠️ Please use the correct wrench supplied with the tip, and make sure it is fully tightened.

Knowing when to replace your tips is essential, as tip wear will significantly reduce scaler performance – as it alters the ultrasonic frequency.

PERIO TIPS - For subgingival and periodontal cleaning needs.



1P
U6PD3



3PI
U6P2DL



3Pr
U6P2DR



4P
U6PD3D



5PI
U6PD2LD

RESTORATION TIPS - For precise dental work.



1R
U6PD4D



3Rm
U6SBD2

ENDO TIPS - For various endodontic procedures.



2E
U6ED14



3E
U640D



5E
U6GD7