

□ ULTRA SCALING TIPS

Knowing when to replace your tips is essential, as tip wear will significantly reduce scaler performance – as it alters the ultrasonic frequency.

PLACE YOUR SCALER TIP OVER THE RELEVANT IMAGE TO CHECK YOUR TIP MEASUREMENT

UNIVERSAL TIPS - Designed for supragingival calculus removal.

Each tip is designed for different cleaning needs, from coarse to fine depuration, and for tackling both large deposits and interdental areas.



1U
U801U

Operating Settings:
Ideal: 30
Max: 40

For coarse depuration of supragingival calculus.



Up to 1 mm:
Optimum efficiency



1-2 mm:
Time to re-order



Over 2 mm:
Time to dispose



1P
U801P

Operating Settings:
Ideal: 10
Max: 30

Suitable for the removal of subgingival deposits.



2U
U802U

Operating Settings:
Ideal: 20
Max: 30

Suitable for fine depuration of supragingival calculus.



3U
U803U

Operating Settings:
Ideal: 20
Max: 30

For removing supragingival calculus, and for interdental cleaning.

PERIO TIPS - For subgingival and periodontal cleaning needs.



3PI
U803PL

Ideal: 10
Max: 30



3Pr
U803PR

Ideal: 10
Max: 30



4P
U804P

Ideal: 25
Max: 30



5P
U805P

Ideal: 25
Max: 30



5PI
U805PL

Ideal: 25
Max: 30



5Pr
U805PR

Ideal: 25
Max: 30

UNIVERSAL TIPS
- For supragingival calculus removal.

RESTORATION TIPS - For precise dental work.



1R
U801R

Ideal: 35
Max: 40



2R
U802R

Ideal: 35
Max: 40



4U
U804U

Ideal: 30
Max: 40

ENDO TIPS - For various endodontic procedures.



1E
U801E

Ideal: 20
Max: 40



2E
U802E

Ideal: 35
Max: 40



3E
U803E

Ideal: 30
Max: 40



4E
U804E

Ideal: 25
Max: 30



5E
U805E

Ideal: 30
Max: 40



6E
U806E

Ideal: 30
Max: 40

👉 Reorder new tips at www.im3vet.eu

⚠ Please use the correct wrench supplied with the tip, and make sure it is fully tightened.